

Chamber Brite is a cleaning and descaling agent designed specifically for cleaning and removal of water deposit oxides and other sediments that are found in steam sterilizers.

The material is a combination of acidic salts and additional cleaning materials. Use 1 prepackaged Chamber Brite tablet for all Tuttnauer Autoclave models*

* Chamber Brite is not for use on Elara 11 or any other pre/post vac autoclaves. Clean every 20 cycles or as needed.

Cleaning procedure:

1. Important- all steps in this procedure must be completed without interruption.
2. When autoclave chamber is cold, remove instruments and trays.
3. Place trays and tray holder in the sink to be cleaned with a stainless steel SAFE cleaner.
4. Fill the reservoir with distilled water.
5. Open the door in any chamber and place the tablet/towards the back of the bottom of the cold chamber.
6. Run a standard unwrapped cycle with 3 minute sterilization time at 273°F

(134°C) with no drying cycle according to the manufacturer's instructions on all Tuttnauer "E/EA/EZ/EZPlus/EKA/EK" series units. All cycles referenced are from a COLD start. When the cycle is finished, exhaust the unit.

7. At the end of the exhaust cycle, drain the water from the reservoir.
8. Refill the water reservoir with distilled water.
9. Repeat step 6, but without using Chamber Brite.
10. At the end of the exhaust cycle, drain the water from the reservoir.
11. Turn the autoclave off and allow the chamber to cool.
12. Wipe the interior of the chamber with a dry cloth.
13. Refill the reservoir with distilled water only.
14. For M/MK/ Valueklave models, turn the "Fill" knob to the fill position and allow a small amount of water (4 ounces) to enter the chamber.
For all: E/EA/EZ/EKA/EK models, press the "Manual Water Fill" button and allow a small amount of water (4 ounces) to enter the chamber.
For all units, remove this water from the chamber
15. The instrument is now clean and ready to use.

IMPORTANT: Do not sterilize instruments during the cleaning process!!!

Caution: Gloves recommended when handling tablets. Keep out of the reach of children. Contains mildly acidic ingredients. Avoid contact with the skin, eyes or clothing.

Wash hands well after touching the tablets. In case of eye contact, flush with continuous running water for at least 15 minutes. If irritation persists, get medical attention.

If accidentally swallowed, do not induce vomiting, drink large amounts of water and obtain medical attention. MSDS available upon request.

To purchase more Chamber Brite, please contact your local dealer

N.B.: Do not open Chamber until the completion of step #10

Chamber Brite

CLEAN & SHINE

Your Autoclave Chamber

**INSTRUCTIONS FOR CLEANING
TABLETOP AUTOCLAVES
WITH CHAMBER BRITE**


Tuttnauer
Innovation · Legacy · Partnership

